



Snug Feet

One unit consists of:

2 Snug feet

4 washers

4 sheet metal screws

Warning

Snug feet should only be used for positioning in a wheelchair. These systems are not safety belts and must not be used in any other application where its failure could result in injury. Snug Feet should only be used on persons wearing a properly fitted hip belt.

Only mount on firm footplates. Snug Feet may not be used without footplates.

They must be fitted and worn over every day footwear- heavily padded or orthotic (devices oder shoes ?) may interfere with their proper fit.

General hints:

1. First, position the foot on the footplate and wrap the Snug Feet around the ankle. Next, position it with the strap on the outside and the belt on inner and outer bone. Tighten the hook and loop strap. The fit should be tight but leaving enough room for one finger between Snug Feet and Achilles tendon. Check the belt's proper fit every time it is worn.
2. Grasp one of the side straps and pull it down until the belt sits exactly on the upper side of the foot. It should now have enough tension as to securely hold and support the ankle joint.
3. Place on washer onto the right strap and mark the spot where it sits on belt and footplate. Repeat steps 1 to 3 to install left strap
4. Return the washers to the spots your marked and carefully drill through the footplate.
5. Attach all straps to the footplate by passing a sheet metal screw through the mounting plate and then lead through the lower strap at the marked spots.

Mounting directions

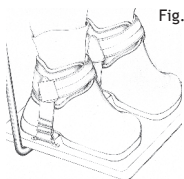


Fig. 1



Fig. 2

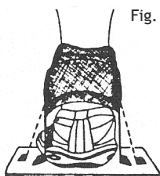


Fig. 3



Fig. 4



Fig. 5

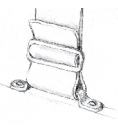


Fig. 6

Mounting Snug Feet

Fig. 1

Correctly installed Snug Feet with hook and loop strap on the outer side of the foot. It may be installed on the inner side as well, when needed (see fig. 7 how to mount the closing strap).

Fig. 2

Cut away view to show position of the ankle.

Fig. 3

Placement and tension of lower straps on footplate allows varying degrees of foot movement.

Fig. 4

Mounting scetch

Fig. 5 and 6

See fig. 3, but with adjusting buckle mounted to foot plate.

Fig. 7

How to mount the closing strap

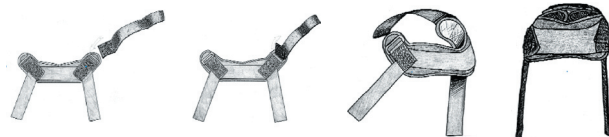


Fig. 7

Snug Feet are meant to secure feet in everyday footwear.